
-⊗-

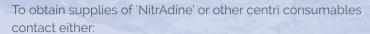

- 1 Your local Dental Surgery
- 2 Dental Appliance Manufacturer

3 WHW Plastics ltd Therm Road Hull HU8 7BF • Tel 01482 329154

Pk of 20 tablets - NitrAdine (10 wks supply)	£	10.00	£
Pk of 32 tablets - Denture Cleaner	£	4.75	£
1 x 45g NitrAdine Shine Tooth Paste	£	5.90	£
1 x Aqueos Sport Gum Shield Disinfectant	£	4.95	£
1 x TCS cleaner (40 wks supply)	£	25.00	£
Dent-Spa Case	£	15.00	£
Olivafix	£	6.75	£
Jefe Tooth Brush	£	5.95	£
Post & Packing	£	1.99	£
Please contact your Dental Surgery or Appli Manufacturer for details on how to make pa GOODS SHOULD ARRIVE WITHIN 14 DAYS	£		

\setminus	la	m	IE
-------------	----	---	----

Address

Postcode Tel No



centri after care

sales 0800 0092 444 head office 01482 329 154



DLA 2015 TRADE

AWARDS
Trade Company Of
The Year

Winner

WHW Plastics Ltd

Therm Road, Hull HU8 7BF East Yorkshire, United Kingdom fax +44 (0)1482 217140 email: sales@whwplastics.com • website: www.whwplastics.com

Registered England & Wales 6942043







NITRADINE

Most removable dental appliances (dentures) are made from porous materials in which opportunistic and pathogenic micro-organisms (fungi, bacteria and viruses) infiltrate, attach and form bio-film, a 3-dimensional protective matrix in which they build-up. Especially, Candida albicans deeply penetrates the pores and cracks of the dental appliance. Also other pathological micro-organisms such as Pseudomonas St mutans, St aureus (inc MRSA), aeruginosa and Herpes simplex can attach to and colonise the appliance which leads to painful irritation of the oral cavity (denture stomatitis).

Elderly, diabetic, transplant and cancer therapy patients wearing contaminated appliances can develop serious systemic infection of the upper airways. For example, oropharyngeal candidiasis is observed in 75% of denture wearing patients in nursing homes. Also the lungs, heart and kidneys can become infected which in some cases may lead to organ failure.



Clinical trials have shown that NitrAdine tablets result in a 99.995% bio-film reduction (Bacteria).

NitrAdine tablets or solution are not to be taken orally.

SUITABLE FOR FLEXIBLE DENTURES

NitrAdine tablet patented ingredients.

Citric Acid, Sodium Lauryl Sulfate, Lactose Monohydrate, Sodium Bicarbonate, Sodium Chloride, Potassium Hydrogen Monopersulfate, Peppermint flavour, PVP.

DENT SPA

Denture box with ultrasonic vibrating action for removable appliance cleaning.

For best results use 'NitrAdine' tablets at least twice a week to leave your appliance fresh, clean and disinfected against harmful bacteria. If you have symptoms of dental thrush use tablets daily for two weeks to eliminate the bacterial build-up.

Empty Dent-Spa after use, wipe clean, **DO NOT submerse or**

rinse under running water as this will damage the unit.



JEFE SILICONE DENTURE BRUSH

Unlike a regular brush, the soft silicone bristles are specifically made for dentures and flexible dentures. The Jefe is non abrasive and will help keep your denture in excellent condition.









With its "Anti-biofilm" formula, NitrAdine SHINE has been especially formulated to clean, whiten, refresh and polish your denture or removable appliance. NitrAdine SHINE has been designed to be used on a daily basis as part of your cleansing routine.



AQUEOS SPORT GUM SHIELD DISINFECTANT

Gumshields are essential to safety while playing many sports. They protect the teeth, prevent cuts and accidental bites and may even reduce the chance of getting knocked unconscious.

Mouthguards are often kept in boot or kit bags and only come out just before use. Many are dropped on the floor with the potential for nasty germs to spread or for you to be re-infected after bouts of cold and flu.

Aqueos Sport Gum Shield Disinfectant is mint flavoured and kills 99.999% bacteria, fungi & viruses in less than 30 seconds.

- Safe to use on all mouthwear and containers without causing distortion or degradation
- · Powerful but safe
- Same hazard rating as distilled water
- · No alcohol or bleach
- Environmentally friendly



PROFESSIONAL DENTURE CLEANING TABLETS

Our professional effervescent denture cleanser tablets are simple, efficient and easy to use. The combination of the active ingredients removes plaque, impurities, food remains, nicotine, tea, coffee and other stains.

Brushing your dentures alone, is not as effective as a combination of soaking the denture in the effervescent tablets before brushing. By simply brushing, it is easy not to spend enough time cleaning and it is also difficult to clean the hard to reach places which in turn creates nesting places for bacterial growth.

By brushing your denture with our NitrAdine Shine Toothpaste and soaking your denture in a special denture solution, you will be assured of giving yourself the best denture hygiene possible.

Are you a NitrAdine user?
Combine the daily cleaning tabs with a treatment of NitrAdine for the ultimate protection.



5





TCS FLEXIBLE DENTURE CLEANER

10 month supply

The perfect companion to your new flexible denture, the TCS flexible cleaner - CLEANS, DISINFECTS & REFRESHES!

Used daily, it will keep appliances looking and feeling like new!
Dissolve away stains that brushing leaves behind. Remove coffee, tea, tobacco, and iron stains as well as plaque, tartar, calcium and calculus. It also kills germs that cause bad breath.

Keeps the appliance fresh and clean

Soak appliance for at least 15 minutes per day. Appliances can also be left soaking overnight or whenever they are not being worn, except appliances containing soft liners, these should not be soaked for more than 15 minutes per cleaning.





OLIVAFIX

A denture fixative cream like no other. What is so special about this product? It's simple. It is the first denture fixative cream on the market that contains NO Zinc & NO Petrolatum (more commonly referred to as "Vaseline"). The Zinc and Petrolatum have been replaced by almost 35% Olive Oil. Combining this with a strong, long lasting hold for up to 24 hrs and the fact that Olivafix contains no added preservatives proves that Olivafix is the name for the future for Denture Fixative Creams.

Olivafix is hypoallergenic and leaves a silky feeling in the mouth. It has been clinically

tested by consumers with 100% satisfaction

and is compatible with all types of dentures. Olive Oil is known for its anti-bacterial and Anti-Candida properties, making it the ideal choice for consumers.

- Strong 24 hour hold on full upper denture
- · 0% Zinc
- 0% Petrochemicals
- 0% Parabens
- Silky smooth texture
- Neutral taste
- · Up to 35% olive oil



Innovative denture fixative cream