# LASCOD

**Zeffiro™**

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168 HOURS  
**EDS**  
EXTRA  
DIMENSIONAL  
STABILITY

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DUST FREE - ISO 1563  
Class A - Type I - FAST  
MADE IN ITALY

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*Discover all the new items highlighted in  
the catalog with the symbol.*

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*Stay updated on the last Zeffiro news  
by connecting at: **news.lascod.it**  
or by activating the Qr code on your  
smartphone.*

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# 1946: a

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## 1946: in Florence an idea was born.

*1946: Italy became a republic, and the first "Vespas" started to circulate, soon becoming one of the symbols of the "economic boom". During the same year in Florence, a dental depot representative came a day to visit a university professor with the intent of identifying and obtaining a dental cement formulation. He wanted to introduce to the market a dental cement with the same specifications of a small sample of reference. Once the formulation was discovered, they faced the challenge of understanding the manufacturing process. Thanks to experience, commitment, passion and a bit of luck the professor was able to produce a dental cement of a much better quality than the one received as sample. A small laboratory was then established in 1946 and later transformed in a dynamic international chemical company.*

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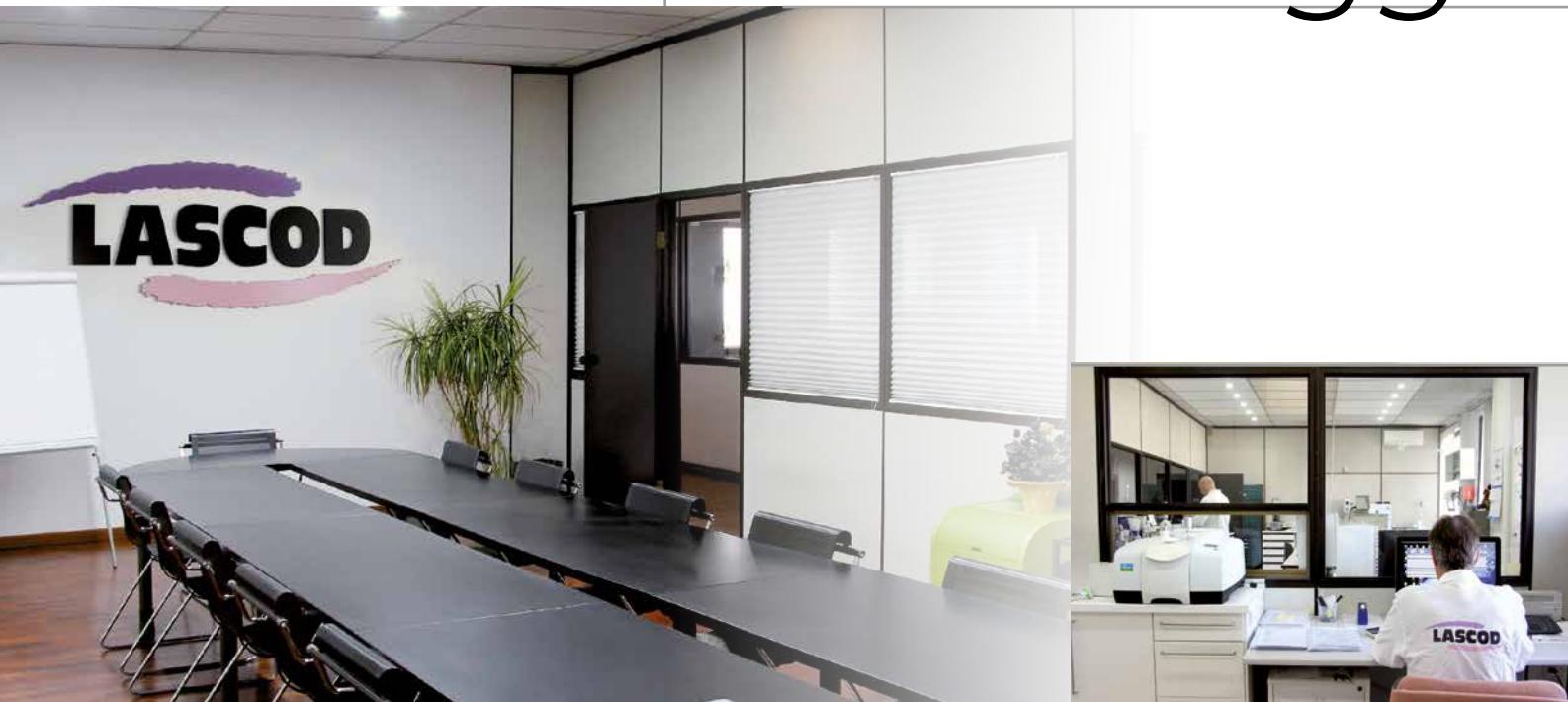
*Some images of the first Lascod range of production headquarters in Via Sirtori, Florence.*

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# importante.

## Lascod today: a great goal.

70 years of marketing experience. Highly skilled personnel, an extensive sales network reaching all over the world, product quality recognition. All that, together with the most modern manufacturing, packing and logistics system have made of LASCOD products a reference point in the dental field. Our knowhow and success in the dental field also allowed us to explore other fields of application such as podiatry, audiology, cosmetics, movie and mechanical industry. We strive for innovation and change while remembering our tradition to achieve a sustainable development of products and technologies. New products and market development, personnel continuous training in and outside LASCOD are our tools to achieve excellence. Our customers will always be the reason of our success.

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CE DUST FREE - ISO 1563  
Class A - Type I - FAST  
MADE IN ITALY

168 HOURS  
EDS  
EXTRA  
DIMENSIONAL  
STABILITY

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LASCOD

**168 HOURS  
EDS  
EXTRA  
DIMENSIONAL  
STABILITY**

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All LASCOD alginates, maintain a very high dimensional stability through time. LASCOD alginate impressions should be kept in a sealed plastic bag (Safe Lock Bag Lascod) without adding water or wet paper or cotton.

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*Since KROMOPAN was created it has been an incredible break through in dentistry allowing, thanks to the chromatic phase indicator (purple = mixing, pink = tray loading, white = mouth insertion), the impression taking with high quality and easy check through the working phases.*

			<b>Just 30'' in mouth</b>
Type I	WORKING TIME	1'45"	
Type II		2'45"	

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*It has the same features as Kromopan but a longer working time of 60" (tot 2'45").*

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*Chromatic alginate (purple = mixing time, pink = tray loading, green = insertion in the oral cavity) SPECIFIC FOR ORTHODONTIC IMPRESSIONS. It has the highest degree of elasticity and resistance to tear, ideal to highlight details in impressions with significant undercuts and brackets.*

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*New generation alginate and top quality level product specific for high precision impressions with details definition up to **15µm**. Some indicated uses include :*

- mobile prosthesis;
- removable and fixed prosthesis antagonists
- temporary prosthesis
- preliminary models
- metal frameworks
- position impressions

*It's the ideal solution for any use where detail definition is essential for perfect prosthesis realizations.*

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*For better preservation of all LASCOD alginate powders we recommend to use our plastic tub.*

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**SAFE DISINFECTION:** all impressions taken with Lascod alginates can be disinfected with specific products following manufacturers' instructions about dosage/proportions, temperature, submersion timings, solution substitution and disposal without altering the characteristics of the material.

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*Non chromatic alginate with the same technical features and same precision as Kromopan, indicated for the same applications: removable and fixed prosthesis antagonists, temporary prosthesis impressions, preliminary models for metal frameworks and orthodontics.*

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- Fixed dose of the powder.
- Easy to use, no waste of material.
- Saving of time during the preparation phase.
- Perfect powder's preservation (no contamination).
- Ease of storage within the dental practice.

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*The new IQ alginates guarantee the best performance in the dental impression taking by reducing costs to a minimum.*

*IQ is available in two different releases:*

*- Green color*  
*- Color changing (purple = mixing, cyclamen = tray loading, green = mouth insertion)*  
*Both are versatile and easy to use because the long working time allows the Dentist to perform the steps of mixing and charging the impression tray with extreme ease.*  
*The fast setting time in mouth (60") minimizes patient discomfort.*  
*The high elasticity and tear resistance guarantee a perfect reproduction of details and consequently the creation of a very precise dental model.*

- |  |   |
|--|---|
| <ul style="list-style-type: none"><li>• Facili da utilizzare</li><li>• Versatili</li><li>• Tissotropici</li><li>• Elevata definizione dei dettagli (50 µm)</li><li>• Tempo di presa rapido</li><li>• Compatibili con i liquidi disinfettanti</li><li>• Aroma menta</li><li>• Dust free</li><li>• Scadenza 5 anni</li></ul> | <ul style="list-style-type: none"><li>• <i>Easy to use</i></li><li>• <i>Versatile</i></li><li>• <i>Thixotropic</i></li><li>• <i>High definition details (50 µm)</i></li><li>• <i>Fast setting time</i></li><li>• <i>Compatible with disinfectants liquids</i></li><li>• <i>Mint flavor</i></li><li>• <i>Dust free</i></li><li>• <i>Long expiry date (5 years)</i></li></ul> |
|--|---|

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*It is the innovative solution (neutral pH) to remove residues of alginate, plaster and cement normally deposited on impression trays and dental instruments. The residues removal and the correct deter-sion are the basic steps for a correct use of the instruments.*

*Above all that, Lascod has a special consideration for the environmental conservation.*

*The formulation of All Cleaner is based on the new generation of active ingredients enviromentally friendly. Cleaning agents contained in the powder are completely biodegradable.*

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*With the single-use packaging 50g, **largely enough for one liter of solution**, AllCleaner Powder is easy and ready to use in seconds. It also can be used with ultrasonic equipments facilitating the removal of residuals and reducing further the submersion timings.*

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**Temporary filling cement, ready to use and extremely easy to apply. It doesn't contain eugenol.** The product is characterized by high plasticity. LITARK adheres extremely well with the cavity surfaces with no need to add any further retentions. The cavity does not need to be dried before application. LITARK sets quickly on contact with saliva. Litark low solubility ensures impermeable, completely neutral fillings resistant to mastication. Litark is easy to remove and does not leave residues.

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The new packaging in a 30-g tube with screw cap allows for perfect closing after use, thus keeping the characteristics of the product unaltered after opening the tube.

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### privo di eugenolo eugenol free

The improved viscosity ensures better workability, facilitating application.

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**Temporary cement** - A zinc oxide temporary cement for crown, bridges, inlay and onlay cementing. Fillfix excellent viscosity allows easy mixing and an ultra-thin film preparation for easy application. Fillfix zero solubility is a guarantee for stable cementing over time with a low risk of seepage. Easily removable, it doesn't leave any residuals onto the surface of prosthesis preparations. **Available with or without eugenol**.

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**Water based glass ionomer cement**, formulated for deciduous teeth class 1, fillings, repairing cuneiform defects, erosions of enamel and roots at the neck of the tooth, class 3 permanent fillings, class 5 fillings, enlarged fissures, support fillings for crowns and bridges. Strong chemical bond doesn't need adding retentions, so preserving the original tooth structure. Short setting time, easy modelling and high resistance to abrasion and compression make durable every application.

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**Micro fine glass-ionomer water based cement** for permanent cementing of crown, bridges, inlays, onlays, orthodontic bands and cementing of ceramic restorations (silicate ceramic, zircon oxide, aluminum oxide). Easily applicable, no dimensional variations, stable expansion setting, high compression resistance and high solubility grant precision and durability.

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Perfect radiopacity in Kromoglass 2 allows checks easy to read over time.

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High fluoride release in Kromoglass 2 and 3 also contributes to long lasting bonding and sensibly diminishes the risk of new caries.

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**HYDROPHILICITY AND ACCURACY** - Ghenesyl easily adapts to different dental structures and preparation cavities in a moist clinical environment while reproducing fine margins. The contact angle measurement (wettability index) is less than 10°.

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*Polivinilsiloxane addition curing silicone is a versatile and efficient product guaranteeing maximum precision in humid conditions. GHENESYL has different grades of hardness (Hard/Soft) and viscosity (Superlight/Light/Regular/Heavy) which attain the best results adapting to every impression technique. A long working time increases the product workability. At the same time a short setting time in mouth is guarantee for the best patient's comfort.*

	colore	Tempo di lavorazione incluso il tempo di miscelazione - (23°)	Tempo minimo di permanenza in bocca (37°)	Ritorno elastico	Stabilità dimensionale
	color	Working time Including mixing time (23°)	Minimum time in mouth (37°)	Elastic recovery (%)	Dimensional stability
<b>PUTTY HARD</b>	pink	2' 45"	2' 15"	99.9	7 days
<b>PUTTY SOFT</b>	orange	2' 45"	2' 15"	99.9	7 days
<b>SUPERLIGHT</b>	purple	2' 00"	1' 30"	99.9	7 days
<b>LIGHT</b>	light blue	2' 00"	1' 30"	99.9	7 days
<b>REGULAR</b>	teal	2' 00"	1' 30"	99.9	7 days
<b>HEAVY</b>	green	2' 00"	1' 30"	99.9	7 days
<b>MONOFASE</b>	purple	2' 00"	1' 30"	99.9	7 days

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*GHENESYL is available in KIT PACKS containing 2x150ml putty base and catalyst and a cartridge of 50ml wash.*

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15µm

mint flavour

*Ghenesyl Mono, developed by LASCOD, is a new impression material with a medium viscosity, ideal for monophase technique. Recommended for crowns and bridges impression taking, implantoprosthetic, inlay, onlay, partial and total edentulism. Thanks to its characteristic viscosity it can be used directly on the surfaces to be reproduced and on impression tray. It's syringable through the included mixing tips or through the specific syringe for elastomers.*

*Given its optimal flexibility Ghenesyl Mono can be easily removed by the mouth without structural alteration.*

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40% material savings using the pink mixing tips included.

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*It's an addition silicone (polyvinylsiloxane) specific for orthodontic occlusion registration, registration keys for orthognathic registrations, inter-maxillary registration keys for centered positions, eruptions and ectopic eruptions, registrations for cephalometric analysis, which requires a CAD scanning (optical, laser and tactile). High final hardness silicone: 95 Shore-A, tixotropic, imperceptible consistency for the patient with regular occlusion. Dimensionally stable even during disinfection. With easy positioning in mouth or onto models after setting, it can be trimmed with cutters or burs.*

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*Silaxil Putty and Silaxil Light Body, the last technological development of C-silicone at Lascod providing the highest accuracy in details survey. Adaptable to every impression technique. Its high dimensional stability allows preserving the impression over time without any dimensional alteration of the models.*

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*The selection of delicate flavors, mint for the Putty and orange for the Light Body, allows not to alter the degree of the patient's saliva and greatly reduces the emetic effect.*

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*Silaxil is also available in a kit made up of:*

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*It can be used for detailed fast reproductions of gums morphology to survey cervical limits in prothesis manufacturing for implants, crowns, bridges, granting high precision.*

### **It can be scanned with optical/laser/tactile systems.**

*Easily and safely workable thanks to double cartridge system 1:1.*

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*The insulating liquid **XILGUM SEP FLUID**, specially developed for XILGUM silicone, perfectly isolates impression materials and laboratory silicones ensuring minimum thickness without any residual surface for a perfect gingival reproduction.*

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Laboratory silicone for frameworks, temporary prothesis, gum reproductions, (see also XILGUM), ceramic and composites printing, tenons positioning, occlusal registration for CAD scanners, restoration models, copying of plaster models without insulating materials, copying of complete prosthesis, positioning and addition of teeth on prosthesis and metal frameworks, insulating of acrylic teeth in flasking technique.

**It can be scanned with optical/laser/tactile systems.**

*Easy and clean mixing thanks to 1:1 dosage*

*High dimensional stability*

*High details definition 40 micron*

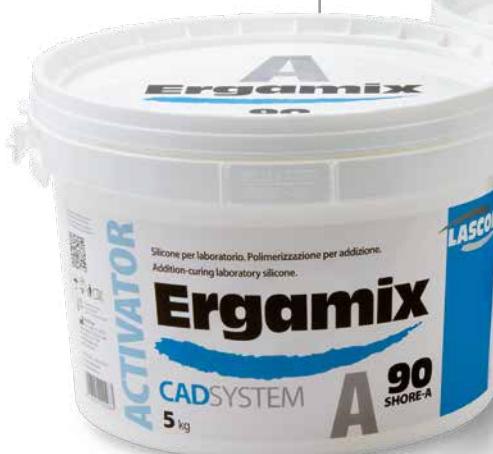
*Final hardness (70 and 90 Shore-A) speedily coming*

*No retracion/deformation*

*Easy finishing by burs*

*Heat resistant even in flasking furnaces*

*Available in brown (70sh) and blue (90sh) color with high cromatic contrast*

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\*misure effettuate a 23°C - \*tested at 23°C.

*It combines a practicality of using with a series of advantages that allow the technicians to obtain the maximum precision in making the prosthetic model, adapting perfectly to all techniques and to all application fields.*

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High accuracy in detailed reproductions (50 $\mu$ )

92 SHORE-A hardness in a very short time

Optimal mix viscosity and workability

High resistance to compression

Excellent adhesiveness to cyanoacrylate

Compatible with self curing and non acrylic resins

Long working time (4 mins)

Can use rotary instruments.

Heat resistant in flaking furnaces

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**ENERSYL** - The new 60 ml. packaging of catalyst for Lascod condensation curing silicone.

For further informations visit us at [www.lascod.com](http://www.lascod.com)

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**An accurate model will enhance the value of the prosthesis and will be the best presentation for a dental laboratory. The model must reproduce the data transmitted by the impression material with the highest possible accuracy.**

**Fast setting time and accuracy** - The micronized and fine powder of Lascod plasters allows the accurate and detailed transfer of the oral cavity data recorded on the impression material, to your model stone. Our plasters/stones are characterized by great strength and surface hardness, strong edges even in thin layers and no splinters during cutting or refining. No breakage risks when separating models from impressions.

Lascod's plasters range is available in three different types of packaging:

- 20/25 kg or 6 kg plastic buckets air-tight lid which is the guarantee for long shelf life with consistent physical properties and quality. Color coded lids ease both stocking and identification of the plaster/stone you need to use.
- 1,25 kg vertical bag

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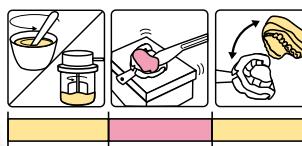
*Master models, crown and bridges stumps, inlay/on-lay, metal frameworks, implants, antagonist.*

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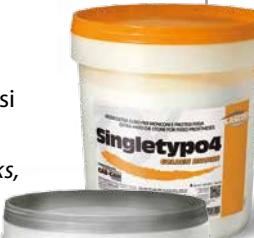
*Master models with complete and partial prostheses, metal frameworks, implants, antagonist.*

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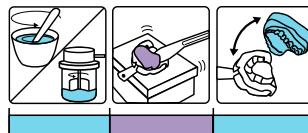
*Antagonist models, Master models with complete and partial prostheses, metal frameworks, study and preliminary models.*

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*Orthodontic, demonstration and antagonist.*

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*Setting on articulator, metal framework, fast setting, low expansion,  
high adhesiveness.*

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*Flasking and creation of full or partially removable prostheses, setting on articulator, metal frameworks, study models.*

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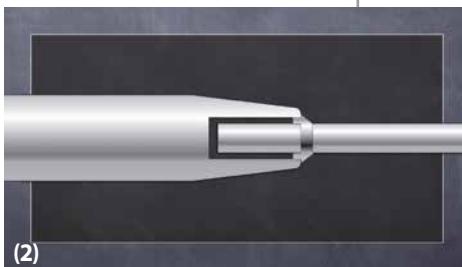
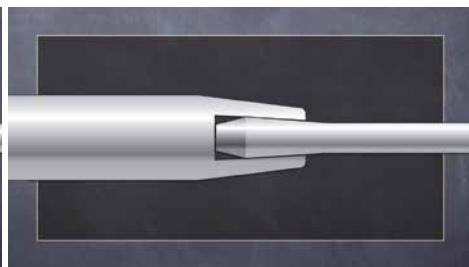
## Absolute hygiene for enhanced safety!

*In its longstanding, customer-oriented commitment, LASCOD has now won another challenge by patenting an innovative surface treatment of surgical steel, giving rise to a new, highly functional smooth handle. This new handle facilitates the rapid removal of any organic or inorganic residues while ensuring excellent grip and control, as well as absolute hygiene.*

### HANDLE-TIP CHONOMETRIC INSERTION

*Besides making the junction site impenetrable, this solution also eliminates the need to add other materials, chemical adhesives (1) or welding (2), which are responsible for triggering and facilitating oxidation and corrosion. University studies have indicated these areas as the most hazardous because of their ability to retain organic material even after thorough cleansing.*

### EASY TO CLEAN / NO KNURLING

*The total absence of any knurled surfaces prevents the use of brushes or abrasive metal scourers that could damage the instrument and that facilitate the onset of corrosion.*

*Manual and ultrasonic cleaning is also very effective, as it facilitates the last rinse and subsequent drying step. The latter is of essential importance in order to avoid compromising the correct and indispensable exposure of the instruments to the sterilising agent.*

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## SECURE GRIP / SENSITIVITY

*The grip of the instrument improves control and operating sensitivity, resulting in less hand fatigue. The handles come in variable diameters: 6 mm (full) and 8 -10 mm (hollow).*

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### ZEFFIRO: RESISTANCE TO CORROSION / LONG LIFE

The Zeffiro instruments are manufactured after selecting and testing the best formulations of surgical steel (in compliance with the UNI EN ISO 7153-1 standard, "Surgical instruments - Metallic materials - Stainless steel") available on the European and world markets in order to ensure maximum resistance to corrosion over time and high quality standards, thus making them suitable for every type of sanitizing and sterilizing technique.

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***Our tangible answer  
to the needs of the most  
demanding professionals!***

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### ZEFFIRO KIT

Convenience and quality come together in the exclusive Zeffiro Kits for use in diagnostics, orthodontic surgery and laboratories.

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On page S/34 we are introducing the new root elevator, the result of multiyear experience accrued at the maxillofacial surgery unit of the University of Florence.

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# "Safeguarding your investment"

## MAINTENANCE AND STERILISATION OF THE INSTRUMENTS.

All the instruments must be washed, disinfected, and sterilized before each use and subsequent re-use. The effectiveness of the washing and disinfection is an essential prerequisite for ensuring proper sterilization.

### GENERAL INSTRUCTIONS:

Always use the following personal protection equipment:

- waterproof gown with elastic cuffs and tight closing at neck;
- airway protection mask;
- cut/puncture resistant gloves;
- protective glasses, preferably a mask (or half-mask of the visor type or equivalent), for protecting the face.

Only use appropriate detergents and disinfectants with demonstrated effectiveness.

Do not use detergent/disinfectant solutions containing chlorine salts, hypochlorite, hydrochloric acid, strong bases ( $pH > 9$ ), strong acids ( $pH < 4$ ), or strong oxidants or peroxides.

Follow the instructions provided by the manufacturers regarding the doses and/or proportions, temperature, immersion times, replacement and disposal of the solution.

Check the quality of the water as this could affect the outcome of the washing and disinfecting operations. The presence of chlorine or other salts in the drinking water could cause the formation of stains and/or corrosion.

Prepare the solutions with demineralized water.

### COLLECTION

Start the collection directly in the operating field using rigid containers suitable for recovery fitted with side handles and an extractable grid.

The instruments used must be subjected to pre-treatment in the fastest possible time after being used (preferably within 2 hours).

It is recommended to use special pincers to pick up sharp instruments.

Sharp disposable instruments (scalpels and needles) must be disposed of in appropriate containers that prevent any risk of exposure to contact.

### DECONTAMINATION

#### Instruments:

- separate the instruments according to the different materials (steel, titanium nitride, tungsten carbide, aluminium, titanium) to prevent any bimetalism phenomena;
- arrange the instruments without any overlapping;
- fully immerse the instruments in the detergent/disinfectant solution;
- OPEN all the instruments with joints and seams (tweezers, scissors, needle holders, etc.);
- use special containers for collecting mirrors and cutters;
- use specific detergents/disinfectants for titanium and aluminium instruments.

#### Containers:

Use appropriate containers in NON-METALLIC material equipped with instrument-holder baskets and fitted with lids and safety extraction devices.

#### ATTENTION:

The disinfectants and operations performed in the preliminary treatment

serve for ensuring personal safety but THEY CANNOT SUBSTITUTE the subsequent disinfecting process.

### WASHING/DISINFECTING

These are the most important steps in the entire process, during which all organic residues must be completely removed to guarantee correct sterilization. Great care must be taken during the RINSING and DRYING steps, because if these are not performed correctly they could invalidate the following sterilization process and damage the instruments.

### MANUAL WASHING

Manually remove impurities, preferably using long-handled brushes with soft nylon bristles.

Avoid exerting excessive pressure and the use of abrasive instruments (metal brushes or scourers).

Fully immerse the instruments in the solution, if necessary carefully wash the less inaccessible parts (cavities, joints, seams, etc.)

Remove the instruments from the solution and rinse thoroughly with demineralized water, inspecting the effectiveness of the cleaning and elimination of all residues of the decontaminating disinfectant.

### ULTRASONIC CLEANING

Do not overload the basket, the instruments should be appropriately spaced apart to prevent damage from excessive contact.

Avoid inserting anodized instruments.

Instruments in different metals should not be washed together at the same time to prevent any bimetalism phenomena.

Avoid including mouth mirrors.

Fully immerse the instruments in the solution.

The device must be fitted with a lid to avoid the formation of aerosol.

The solution should be used at a temperature of between 40°C and 50°C.

It is recommended to fully immerse the instruments for at least 10-15 minutes at a frequency of 35kHz.

Remove the instruments from the solution and rinse thoroughly with demineralized water, inspecting the effectiveness of the cleaning operation.

Wash and disinfect the ultrasonic bath every time the solution is replaced.

### AUTOMATIC WASHING

Use instrument-washing devices with demonstrated effectiveness, compliant with the ISO 15883 standards, and observe the instructions for use indicated by the manufacturer. The instrument-washing devices must be subjected to annual inspections.

### DISINFECTION

Fully immerse the instruments in the solution.

Remove the instruments from the solution and rinse thoroughly with demineralized water.

Carefully dry the instruments one at a time to avoid the formation of superficial marks during the following sterilization step.

Use non-contaminating materials to dry the instruments.

Use paper towels or soft cloths that are fibre-free.

Use compressed air produced by an oil-free compressor (dry) fitted with a dryer filter. Also in this case it is important to use personal protection equipment to avoid inhaling contaminating aerosols.

If heat sources are used to dry the instruments, make sure that the instruments have completely cooled down before continuing with the following steps.

#### INSPECTION / CHECK

Do not use damaged instruments that are broken or show signs of rust, deformation or hairline cracks in the joints.

Check that there are no signs of corrosion that could affect other instruments.

Any instruments with signs of residues of impurities must be washed and disinfected again.

#### LUBRICATION

Lubricate all the instruments with joints and seams (tweezers, scissors, needle holders, etc.).

Use biocompatible lubricants suitable for sterilization in the autoclave (paraffin oil or milk lubricant). Do not use lubricants containing silicone.

Use compressed air to help the lubricant penetrate into the joints.

Check the repeated opening and closing of the lubricated parts.

If necessary, sharpen the instruments, removing any machining residues at the end of the process (sharpening oil and metal particles).

#### PACKAGING

Use bags and rolls of paper coupled with polymer film in compliance with the reference standards.

Avoid putting instruments of different metals into the same bag. Never place too many instruments into the bags as the steam must be free to circulate correctly. It is recommended not to exceed 3/4 of the total volume of the bag.

Instruments with joints (tweezers, scissors, needle holders, etc.) must be opened before being inserted.

Sharp or cutting instruments must be protected to avoid perforating the packaging.

Leave a space of at least 3 cm between the packaging material and the welding. Leave a space of at least 1 cm between the welding and the edge of the paper.

Create a sealed closure with a width of between 6 and 12 mm.

Any printing must be made along the outer edge of the bag beyond the welding.

#### STERILIZATION

It is recommended to use of Class B autoclaves with the fractioned vacuum system.

Never overload the bags in the autoclave baskets.

In order to facilitate the penetration of the steam and the subsequent drying step, the paper side of the bag must be facing upwards (with the transparent side facing downwards).

In case the instruments are not inserted, in bags it is recommended to place a sterilisation-resistant paper sheet between the instruments and the tray.

Use distilled/demineralized water.

Choose the appropriate programme depending on the load and the material used.

The maximum sterilization temperature is 138°C.

Sterilization time (exposure to the sterilization temperature) - a minimum of 15 minutes at 121°C with a pressure of 1.1 bar, or 7 minutes at 134°C with a pressure of 2.5 bar.

The steam used for the sterilization must be free of impurities so as not to compromise the sterilization process or cause damage to the material being treated.

Since the instruments in bags must be completely dry after sterilization, it is recommended to use autoclaves with an automatic drying programme. If any water remains inside the bags it could compromise the sterility conditions and give rise to corrosion and oxidation phenomena.

Any bags which are still wet or opened must be re-processed.

#### STORAGE

Only extract the sterilized instruments once they have completely cooled down.

Ensure that there are no accidental openings in the bags (tears, holes, etc.).

Store in a dust-free and humidity-free area.

Store in a tightly closed and protected environment, if possible at a temperature of between 18°C and 22 C° with a relative humidity between 35% and 50%.

The instruments packed in individual bags will be sterile for 30 days while those in double bags will be sterile for 60 days.

#### TRACKING AND TRACEABILITY

After being sterilized, the following information must be clearly marked on each bag (if possible, on an adhesive label):

- sterilization date and expiry date;
- code of the operator who performed the sterilization;
- progressive number of the cycle;
- type of cycle;
- serial number of the sterilization machine;
- contents.

This information makes it possible to track the location of each single instrument. It is important to create a register that contains all the information and all the documentation regarding the equipment and the products used during the sterilization process.

#### CONTROL AND MAINTENANCE OF THE AUTOCLAVE

All ordinary and extraordinary maintenance operations must be performed and recorded by authorized technicians.

Check the correct functioning of sterilization process via periodical test control.

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# Exploit

*The basic principle of "ABSOLUTE HYGIENE", successfully used for the set of instruments ZEFFIRO, have enabled the creation of EXPLOIT impression trays, made from one single cold-pressed sheet of stainless steel (patent of LASCOD).*

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*EXPLOIT is the only one with adjustable front walls. EXPLOIT is the only one with an anatomical design for utmost accuracy and ergonomic properties to satisfy patient's requirements.*

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*The absence of soldering and retention edges facilitate the removal of residual material which is a breeding ground for blood, spores and bacteria in the mouth.*

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*Perforated and non-perforated EXPLOIT impression trays are compatible with all impression materials and are available with a micro-sand blasted satin finish for even more retention.*

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*EXPLOIT has been designed bearing in mind the oral anatomy and an optimal distribution of the impression material, thus offering greater comfort to the patient.*

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*EXPLOIT impression trays were designed for non edentulous patients, but they are also indicated in total edentulous cases.*

*Four sizes are available for the upper and lower dental arches (extra small - small - medium - large)*

FORME <i>SHAPES</i>	Larghezza <i>Width</i>	Lunghezza <i>Length</i>	Forato <i>Perforated</i>	Non forato <i>Non -perforated</i>
<b>SUPERIORE <i>UPPER</i></b>	mm.	mm.		
<b>XS</b>	<b>62</b>	<b>50</b>	<b>EXF112</b>	<b>EXL312</b>
<b>S</b>	<b>66</b>	<b>53</b>	<b>EXF113</b>	<b>EXL313</b>
<b>M</b>	<b>69</b>	<b>57</b>	<b>EXF114</b>	<b>EXL314</b>
<b>L</b>	<b>72</b>	<b>60</b>	<b>EXF115</b>	<b>EXL315</b>
<b>INFERIORE <i>LOWER</i></b>	mm.	mm.		
<b>XS</b>	<b>60</b>	<b>49</b>	<b>EXF212</b>	<b>EXL412</b>
<b>S</b>	<b>72</b>	<b>53</b>	<b>EXF213</b>	<b>EXL413</b>
<b>M</b>	<b>75</b>	<b>57</b>	<b>EXF214</b>	<b>EXL414</b>
<b>L</b>	<b>78</b>	<b>60</b>	<b>EXF215</b>	<b>EXL415</b>

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*EXPLOIT is also available in kit of 6 pieces, three upper and three lower impression trays, versions:*

**EXK603** - (S - M - L) - satinato forato - *perforated satin finish*

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**EXK605** - (XS - S - M) - satinato forato - *perforated satin finish*

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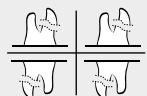
ZFW011 Porta aghi # CW15T - Crile-Wood- Per aghi: 3-0; 4-0; 5-0; 6-0  
Needle holder # CW15T - Crile-Wood- For needles: 3-0; 4-0; 5-0; 6-0  
cm. 15

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*The kit consists of 10 surgical forceps for tooth extraction that can be personalised to meet the dentist's operating requirements.*

di  
agn  
os  
tici

diagnostic

sonde - punta singola | explorers - single-end

**ZFA001**  
# 2

**ZFA002**  
# 23

**ZFA009**  
# 6A

**ZFA003**  
# 9

**ZFA010**  
# 8

**ZFA004**  
# 6

**ZFA012**  
# 12

**ZFA005**  
# 11

**ZFA013**  
# 17

sonde - punta doppia | explorers - double-end

**ZFA020**  
# 23/17

**ZFA025**  
# DG16

**ZFA027**  
# ODU 11/12  
punte extra sottili - extra-thin tips

**ZFA029**  
# TU 17A/23

sonde parodontali - punta singola | probes - single-end

Misure indicative in mm. - Indicative measurements in mm.

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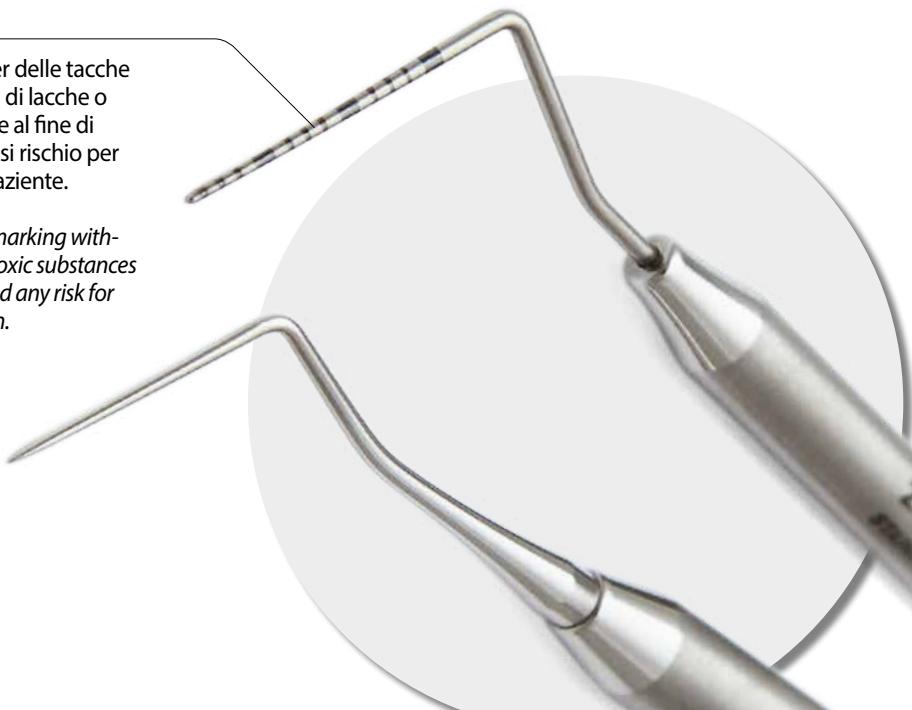
Notches laser marking with-  
out the use of toxic substances  
in order to avoid any risk for  
patient's health.

**sonde parodontali - punta doppia | probes - double-end**

Misure indicative in mm. - *Indicative measurements in mm.*

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*Handles and mirrors are equipped with european standard thread.*

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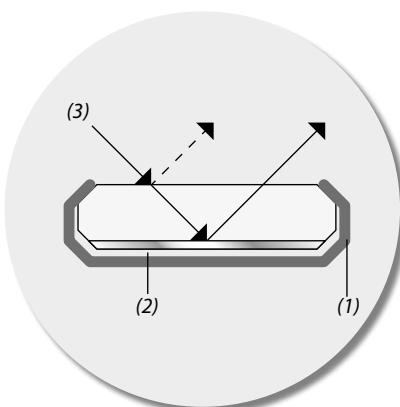
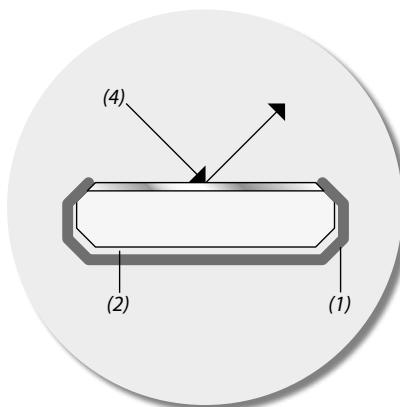
*Mouth mirrors are made of top quality stainless steel and mirrors. On the back, the handle welding was flattened to prevent any possible damage during use in the mouth. In mirrors with aluminum-coated back surface, the reflective surface is at the bottom of the mirror, thus allowing maximum reflection.*

*In mirrors with rhodium-coated front surface, the reflective surface is on the upper part of the mirror for precise and sharp reflected images. It is recommended to avoid contact between the mirror and abrasive surfaces. When cleaning, it also is recommended to avoid immersing the mirror in an ultrasonic tank.*

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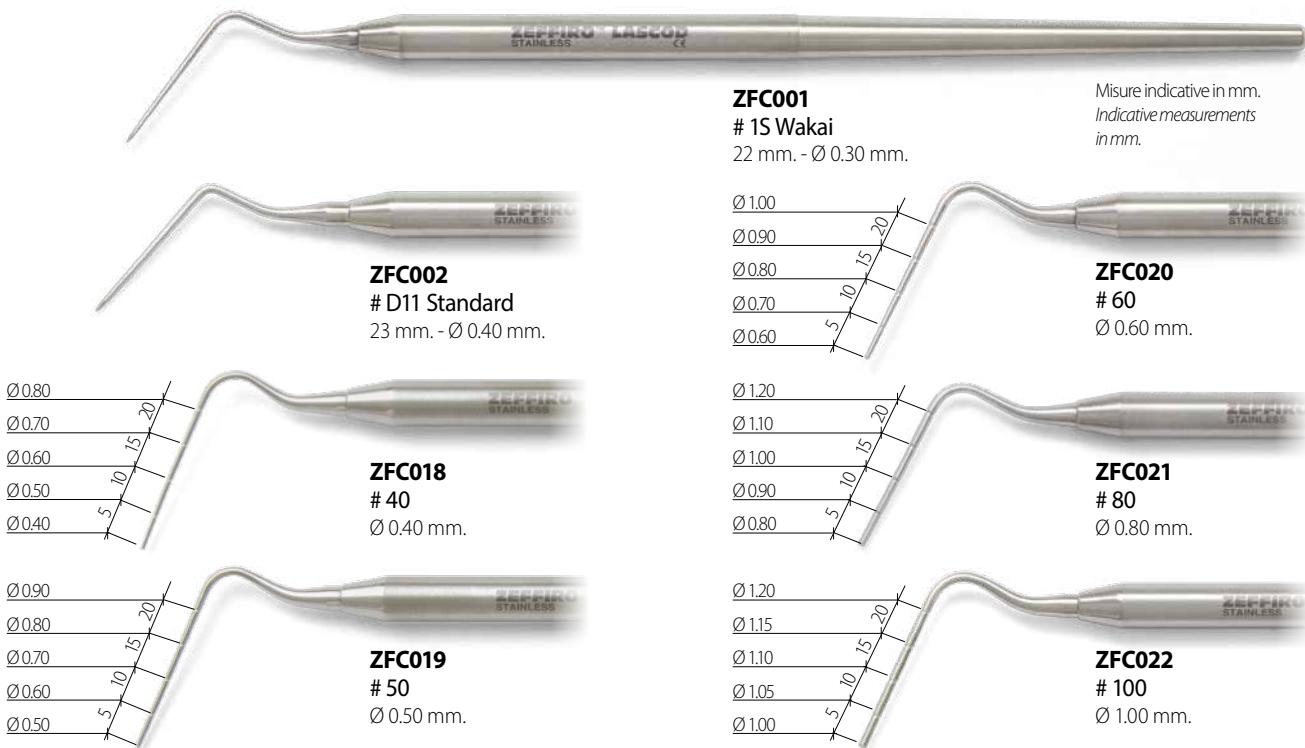
● Misure indicative in mm. - *Indicative measurements in mm.*

● ●  
1.20 1.50

**ZFG004**  
# 1CL

● ●  
1.80 2.10

**ZFG005**  
# 2CL

● ●  
2.40 2.70

**ZFG006**  
# 3CL

● ●  
1.10 1.10

**ZFG021**  
# 0 CL

● ●  
1.20 1.50

**ZFG024**  
# 1 CLS

Zigrinato - Serrated

● ●  
1.80 2.10

**ZFG025**  
# 2 CLS

Zigrinato - Serrated

● ●  
2.40 2.70

**ZFG026**  
# 3 CLS

Zigrinato - Serrated

# conservativa restorative

## brunitori per amalgama | amalgam burnishers

● Misure indicative in mm. - *Indicative measurements in mm.*

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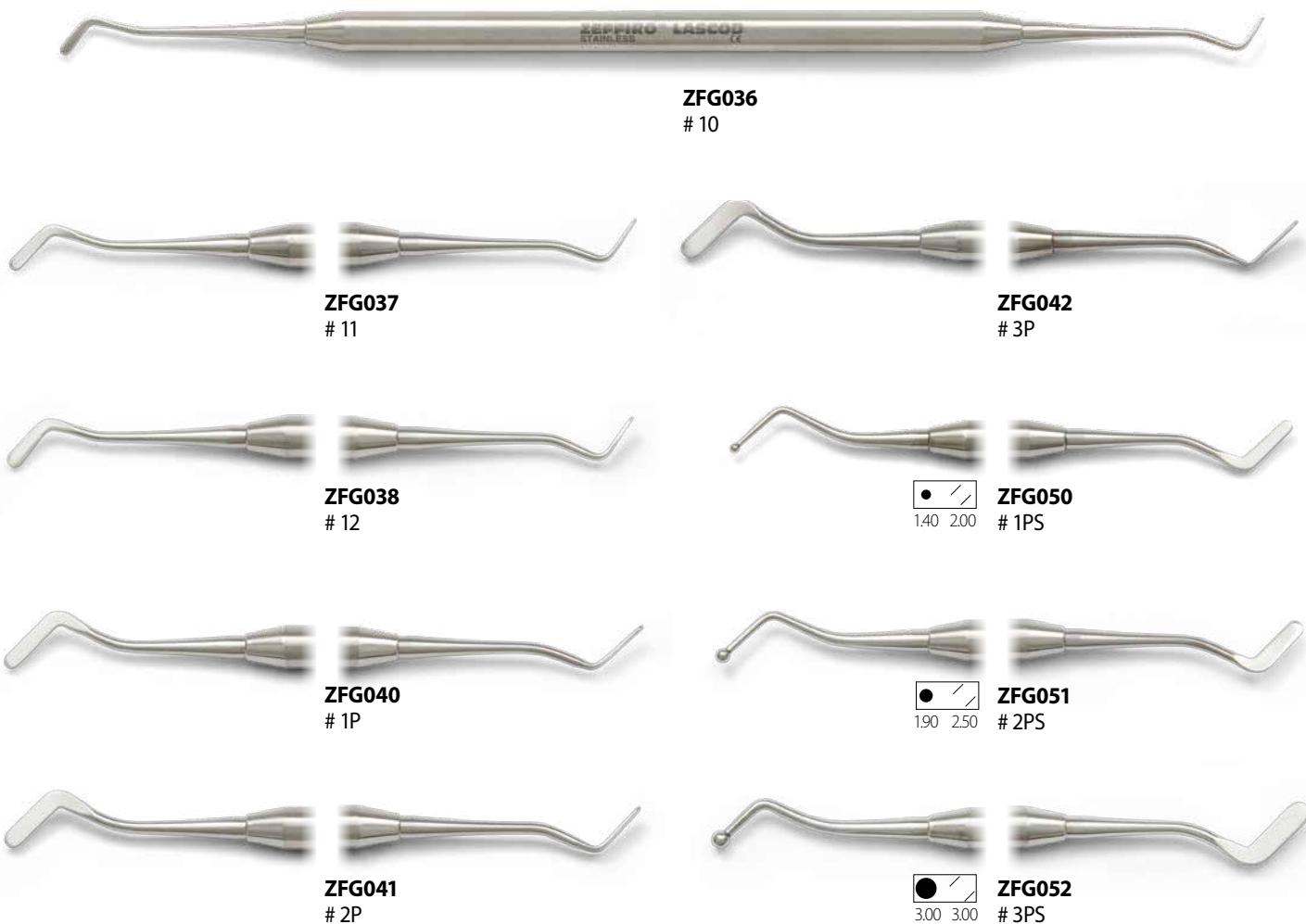
Manico Ø 6 mm. - Handle Ø 6 mm. ● Misure indicative in mm. - Indicative measurements in mm.

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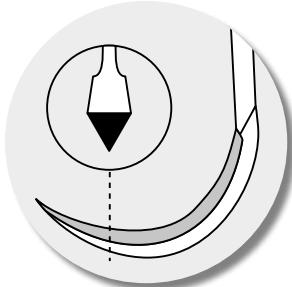
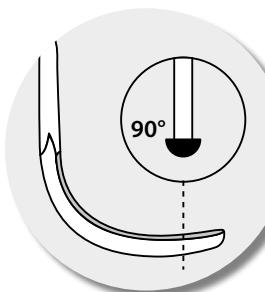
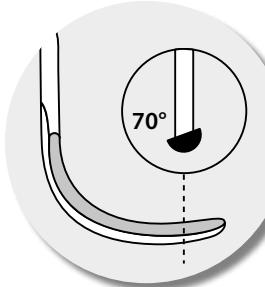
**Straight and curved sickle scalers** feature:  
 - 2 cutting edges  
 - curved facial component  
 - tipped ends

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**Universal curettes** are characterized by an angle of 90° between the terminal stem and the cutting component.

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**Gracey curettes** are characterized by an angle of 70° between the terminal stem and the cutting component.

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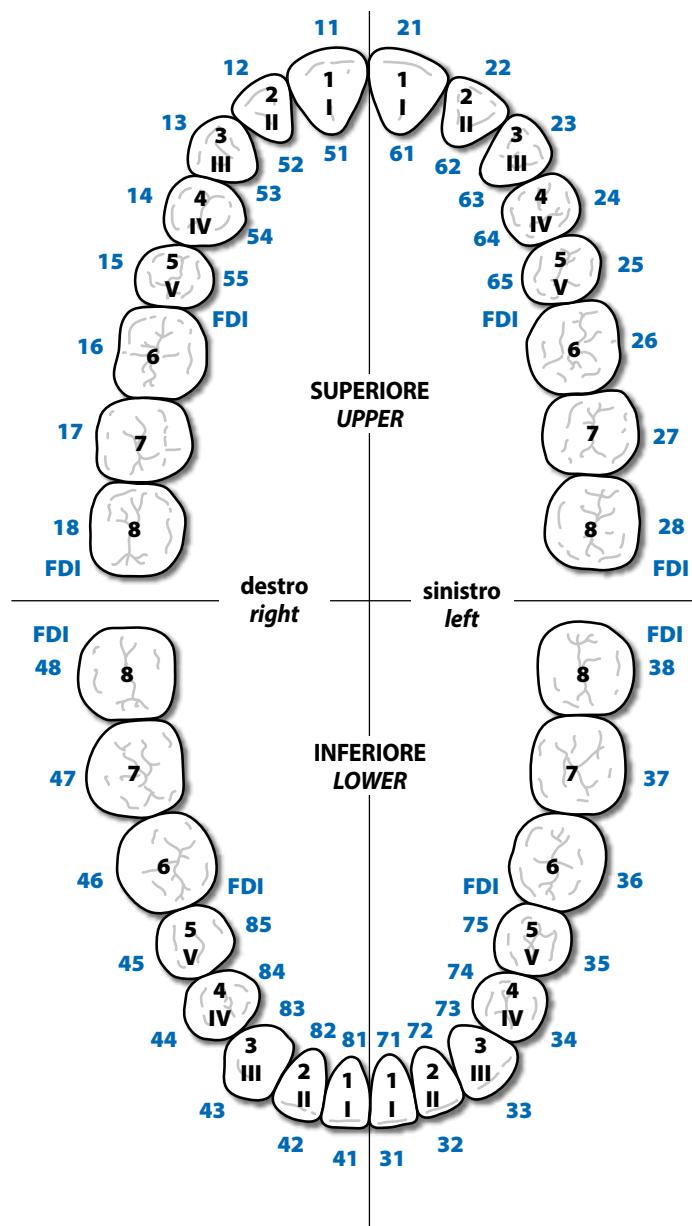
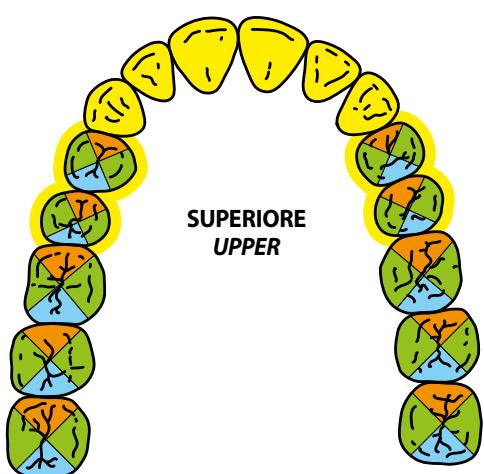
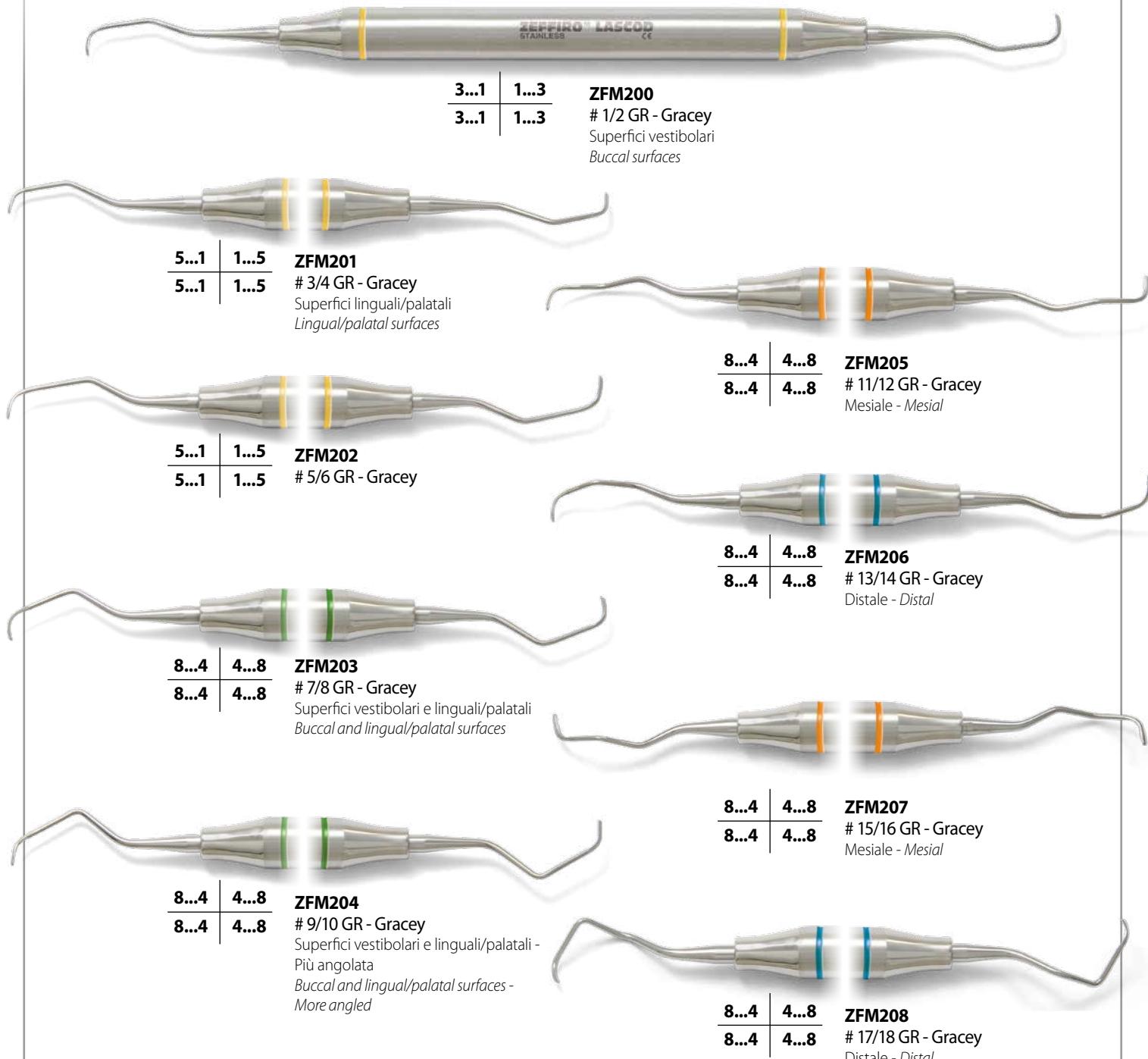
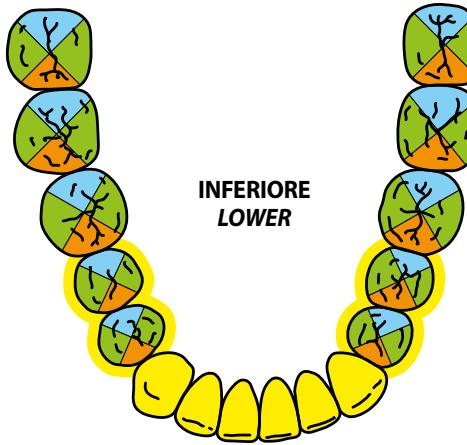
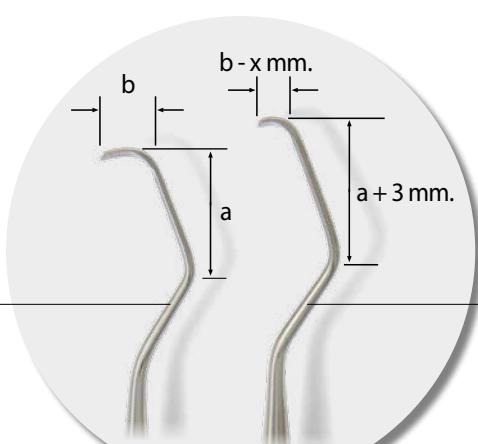
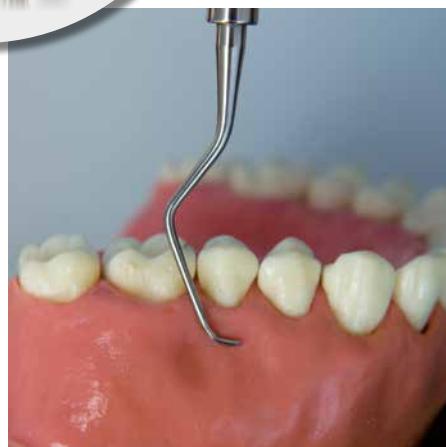
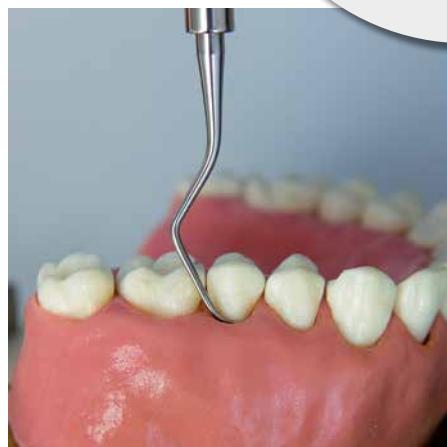
Sharpening stones by LASCOD are to be used for manual sharpening. As natural stones they may present veins and variations in colour. Use medium-grain stones to sharpen blunt instruments or incorrectly sharpened instruments. After using medium-grain stones, then polish instrument with fine-grain stones. Use fine-grain stones to preserve the sharpening or remove small burrs. Using a few drops of oil on the stone while sharpening is recommended.

**ZFM900**  
# 6A - Arkansas  
2/7 x 40 x 100 mm  
Pietra Arkansas grana extra fine (1200)  
Arkansas stone extra fine grain (1200)

**ZFM901**  
# AS22 - Arkansas  
0,5/5 x 25 x 60 mm  
Pietra Arkansas grana extra fine (1200)  
Arkansas stone extra fine grain (1200)

**ZFM902**  
# 6 - Corindone  
2/10 x 50 x 110 mm  
Pietra India grana media (220)  
Indian stone medium grain (220)

pietre per affilatura | sharpening stones

**ZFM903**

# 1 - Tiger

7 x 32 x 80 mm

Pietra naturale grana media (250)

Natural stone medium-grain (250)

**ZFM904**

# 2 - Tiger

7 x 32 x 80 mm

Pietra naturale grana fine (500)

Natural stone fine-grain (500)

**ZFM905**

# 3 - Arkansas

6 x 25 x 85 mm

Pietra Arkansas grana extra fine (1200)

Arkansas stone extra fine grain (1200)

**ZFM906**

# 4 - Arkansas

Ø 8 x 100 mm

Pietra Arkansas cilindrica grana extra fine (1200)

Cylindrical Arkansas stone extra fine grain (1200)

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**MAINTENANCE OF SHARPENING STONES**

During the sharpening of the instruments the removed metal debris may clog the pores of the stones. It is recommended to clean the stones periodically by rubbing of a few drops of lubricating oil upon the surface of the stone. Remove the metal debris with dry gauze. It is recommended to clean the stones with ultrasonic or with a non-metallic brush before carrying out any sterilization procedures.

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